CHORUS

TSH

DIESSSE

DIESSSE

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INSTRUCTIONS FOR USE

CHORUS TSH

For the quantitative determination of human thyroid-stimulating hormone (hTSH)

For *In Vitro* Diagnostic Use Only

1. INTENDED USE

Immunoenzymatic method for the quantitative determination of human thyroid-stimulating hormone (hTSH) in human serum, using a disposable device applied on the Chorus TRIO instruments.

2. INTRODUCTION

Human thyroid-stimulating hormone (hTSH) is a glycoprotein of about 30 KDa, secreted by the anterior pituitary gland. It consists of 2 non-covalently bound subunits (α and β).

The main function of hTSH is to determine the thyroid development, stimulating and regulating the secretion of metabolically active hormones: Thyroxine (T4) and Triiodothyronine (T3). The synthesis of hTSH is controlled by the hypothalamic regulatory hormone (TRH) and by the concentration of the circulating thyroid hormone, through negative feedback.

The hTSH serum level is considered one of the best indicators of the thyroid functionality. The balance, present in normal conditions, among the circulating thyronines level, TRH and hTSH can be altered in several cases of thyroid pathology. High levels of hTSH are found in case of: primary hypothyroidism (from thyroid alteration – low concentration of circulating thyroid hormones); secondary hyperthyroidism (from pituitary alteration – presence of pituitary tumor). Low levels of hTSH are found in case of: primary hyperthyroidism (high concentration of T3 and T4); secondary or tertiary hyperthyroidism, due to rare hypothalamic or pituitary abnormalities.

3. PRINCIPLE OF THE METHOD

The Chorus TSH device is ready to use for the detection of human thyroid-stimulating hormone (hTSH), in the Chorus TRIO instruments.

The test is based on the ELISA principle (Enzyme linked Immunosorbent Assay). During two different incubations, the TSH present in the sample is bound to two different anti-TSH monoclonal antibodies: one present in the solid phase and the other one conjugated with Biotin. After washing to eliminate the components which have not reacted, incubation is performed with Streptavidin conjugated to horse radish peroxidase.

The enzymatic reaction is subsequently stopped by adding the Stop Solution that turns the solution into yellow.

The colour which develops is proportional to the concentration of hormone present in the examined serum.

The disposable devices contain all the reagents to perform the test in the Chorus TRIO instruments.

The results are expressed in $\mu\text{IU/ml}$.

4. WARNINGS AND PRECAUTIONS

FOR *IN VITRO* DIAGNOSTIC USE ONLY

This kit contains materials of human origin which have been tested and gave a negative response by FDA-approved methods for the presence of HBsAg and for anti-HIV-1, anti-HIV-2 and anti-HCV antibodies. As no diagnostic test can offer a complete guarantee regarding the absence of infective agents, all material of human origin must be handled as potentially infectious. All precautions normally adopted in laboratory practice should be followed when handling reagents and samples.

Waste disposal: serum samples, calibrators and strips, once used, must be treated as infectious residuals and eliminated according to law.

Health and Safety Information

1. Do not pipette by mouth.
2. Wear disposable gloves and eye protection while handling specimens.
3. Wash hands thoroughly after placing the devices in the Chorus TRIO instrument.
4. Consult the relative Material Safety Data Sheet (available on request) for all the information on safety concerning the reagents contained in the kit.
5. Neutralized acids and other liquid waste should be decontaminated by adding a sufficient volume of sodium hypochlorite to obtain a final concentration of at least 1%. A 30 minute exposure to 1% sodium hypochlorite may be necessary to ensure effective decontamination.
6. Spillage of potentially infectious materials should be removed immediately with adsorbent paper tissue and the contaminated area swabbed with, for example, 1% sodium hypochlorite before work is continued. Sodium hypochlorite should not be used on acid-containing spills unless the spill area is first wiped dry. Materials used to clean spills, including gloves, should be disposed of as potentially biohazardous waste. Do not autoclave materials containing sodium hypochlorite.

Analytical Precautions

Bring the devices to room temperature (18-30°C) before use; use within 60 min.

1. **Discard devices which show the substrate (well 4) blue colored.**
2. Adding the sample into the well verify that it is perfectly distributed on the bottom.
3. Check for the presence of the reagents in the device and that the device is not damaged. Do not use devices which are lacking a reagent and/or present foreign bodies in the reaction well when visually inspected.
4. The devices are for use with the Chorus/Chorus TRIO instrument; the Instructions for Use must be carefully followed and the Instrument Operating Manual must be consulted.

The use of the kit is only possible with an updated version of software. Make sure that the software installed in the instrument corresponds or has a Release (Rel.) subsequent to the one reported in the

table published on Diesse website
(<http://www.diesse.it/en/Support/Download/strumento:39/>)

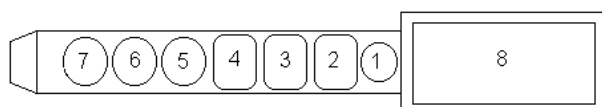
5. Check that the Chorus TRIO instrument is set up correctly (see Operating Manual).
6. Do not alter the bar code placed on the handle of the device in order to allow correct reading by the instrument.
7. Avoid using self-defrosting freezers for the storage of the samples.
8. Defective barcodes can be inserted manually in the instrument (see Operating Manual).
9. Do not expose the devices to strong light or to hypochlorite vapors during storage and use.
10. The use of strongly hemolyzed, lipemic, icteric samples, of serum not completely coagulated or of samples presenting microbial contamination may all constitute sources of error.
11. Do not use the device after the expiry date.
12. **Make sure that the instrument is connected to the Washing Buffer Autoimmunity (Ref. 86004).**

5. KIT COMPOSITION AND REAGENT PREPARATION

The kit is sufficient for 36 tests.

DD DEVICES 6 packages each containing 6 devices

Description:

Position 8: Space for application of bar code label

Position 7: STREPTAVIDIN-POD

Contents: Streptavidin conjugated to peroxidase, in phosphate buffer solution containing phenol 0.05% and Bronidox 0.02%.

Position 6: MICROPLATE WELL

coated with anti-TSH monoclonal antibody

Position 5: MICROPLATE WELL

coated with anti-TSH monoclonal antibody

Position 4: TMB SUBSTRATE

Contents: Tetramethylbenzidine and H₂O₂ stabilized in buffer

Position 3: STOP SOLUTION

Contents: sulphuric acid solution 0.3 M

Position 2: CONJUGATE

Contents: anti-TSH human monoclonal antibodies labelled with Biotin, in phosphate buffer containing preservative.

Position 1: EMPTY WELL

in which undiluted serum must be added

Use: equilibrate a package at room temperature, open the package and remove the required devices; replace the others in the bag with the silica gel, expel the air and seal by pressing the closure. Store at 2-8°C.

CALIBRATOR CALIBRATOR 1 x 0.500 mL

Contents: Diluted human serum containing hTSH and preservative.

Liquid, ready for use.

CONTROL + POSITIVE CONTROL 1 x 1 mL

Contents: Diluted human serum containing hTSH and preservative.

Liquid, ready for use.

MATERIALS REQUIRED BUT NOT PROVIDED

- WASHING BUFFER AUTOIMMUNITY **REF** 86004
- CLEANING SOLUTION 2000 **REF** 83609
- SANITIZING SOLUTION **REF** 83604 - 83608
- CHORUS NEGATIVE CONTROL/SAMPLE DILUENT **REF** 83607
- Chorus TRIO Instrument
- Distilled or deionised water
- Normal laboratory glassware: cylinders, test-tubes etc.
- Micropipettes for the accurate collection of 50-200 µl solution
- Disposable gloves
- Sodium Hypochlorite solution (5%)
- Containers for collection of potentially infectious materials

6. STORAGE AND STABILITY OF REAGENTS

Reagents must be stored at 2/8°C. In the case of storage at an incorrect temperature the calibration must be repeated and the run validated using the control serum (see section 9, Test validation).

The expiry date is printed on each component and on the kit label.

Reagents have a limited stability after opening:

DEVICES	8 weeks at 2/8°C
CALIBRATOR	8 weeks at 2/8°C
POSITIVE CONTROL	8 weeks at 2/8°C

7. SPECIMEN COLLECTION AND STORAGE

The sample is composed of serum collected in the normal manner from the vein and handled with all precautions dictated by good laboratory practice.

Possible consequences, in case of use of other biological liquids, are not known.

The fresh serum may be stored for 2 days at 2/8°C, or frozen for longer periods at -20°C.

The sample can be thawed only once.

Do not keep the samples in auto-defrosting freezers. Defrosted samples must be shaken carefully before use.

Heat-inactivation can rise to erroneous results. The quality of the sample can be seriously affected by microbial contamination which leads to erroneous results.

8. ASSAY PROCEDURE

1. Open the package (on the side containing the pressure-closure), remove the number of devices required and seal the rest in the bag after expelling the air.
2. Check the state of the device according to the indications reported in chapter 4, Analytical Precautions.
3. Dispense 150 µl of undiluted test serum in well no. 1 of each device; at each change of batch, use a device for the calibrator.
4. Place the devices in the Chorus TRIO instrument. Perform the calibration (if necessary) and the test as reported in the instrument Operating Manual.

9. TEST VALIDATION

Use the control serum to check the validity of the results obtained. It should be used as reported in the instrument Operating Manual. If the instrument signals that the control serum has a value outside the acceptable range, the calibration must be repeated. The previous results will be automatically corrected.

If the result of the control serum continues to be outside the acceptable range, contact the Scientific Support.

Tel: 0039 0577 319554
 Fax: 0039 0577 366605
 email: scientificsupport@diesse.it

10. INTERPRETATION OF THE RESULTS

The Chorus TRIO instrument expresses the result in $\mu\text{IU/ml}$, calculated on the basis of a lot-dependent graph stored in the instrument.

The test on the examined serum can be interpreted as follows:

Hyperthyroid	<0.40
Euthyroid	0.40 - 4.4
Hypothyroid	>4.4

11. LIMITATIONS

All the values obtained require a careful interpretation that must consider other indicators relative to the patient.

The test, indeed, cannot be used alone for a clinical diagnosis and the test result should be evaluated together with the patient history and other clinical diagnostic evaluation.

12. CALIBRATION RANGE

Calibration range 0.10-55.0 $\mu\text{IU/ml}$.

For samples > 55.0 $\mu\text{IU/ml}$ retest the diluted sample in the Negative Control/Sample Diluent (PF83607-not supplied with the kit).

13. REFERENCE RANGE

Among the normal population the expected values, which have been determined by examining 158 sera from healthy donors, were between 0.44 and 4.1 $\mu\text{IU/ml}$.

14. ANALITICAL SPECIFICITY

A study has been conducted using the kit reagents with samples containing LH, FSH, PRL, GH and HCG. The presence of these substances in the examined serum does not alter the test results.

15. METHOD COMPARISON

In an experimentation 223 samples have been tested with Diesse kit and with a competitor kit.

Data are summarized in the following table:

Correlation	r	95%CI
Pearson	0.98	0.97-0.98
Spearman	0.95	0.94-0.96

The correlation between the two methods is very high.

16. PRECISION AND REPEATABILITY

Sample	Within run		Between run	
	Mean ($\mu\text{IU/ml}$)	CV%	Mean ($\mu\text{IU/ml}$)	CV%
1	1.00	9.0	0.71	14.1
2	2.0	2.0	1.9	11.6
3	13.4	2.2	13.4	6.3
4	3.2	3.4	0.13	15.4*

Sample	Between lots		Between Instruments	
	Mean ($\mu\text{IU/ml}$)	CV%	Mean ($\mu\text{IU/ml}$)	CV%
1	0.92	7.6	0.92	12.0
2	2.3	9.1	2.3	6.5
3	13.8	8.0	13.8	13.4
4	3.4	13.2	3.4	7.6

* artifact caused by the known fault of Variation Coefficient which becomes extremely sensitive to even very small changes in the mean when the mean value is near zero

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